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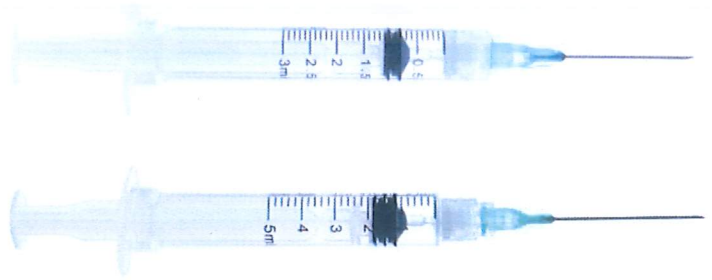
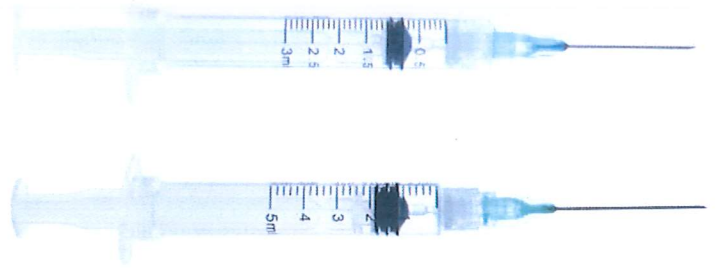
Technical Data Sheet

Product specification

1. Product name SOL-CARE™ Luer Lock Safety Syringe with Exchangeable Needle
2. Description The SOL-CARE™ Luer Lock Safety Syringe with Exchangeable Needle is used to inject medicines and vaccines into, or withdraw fluids from, the body. The Luer Lock Safety Syringe with Exchangeable Needle is designed to aid in the prevention of needle stick injuries.
3. Characteristics After use, the health care professional fully depresses the plunger to engage the Adapter Needle assembly. Once the Adapter Needle Assembly is engaged, pulling back the plunger causes the Adapter and the attached needle to be withdrawn into the safety of the barrel. In this position against the flange, lateral pressure on the plunger results in a controlled fracture of the plunger. Both the syringe and plunger are discarded in a Sharps container.
4. Intended use The Safety Syringe is used to inject fluids into, or withdraw fluids from, the body. In addition, the Safety Syringe is designed to aid in the prevention of needle stick injuries.
5. Instructions for use See IFU (11010103-IFU-003, Revision C)

6. Sizes and REF numbers

REF	Size
100060IM	3ml 21G*1 1/4"
100062IM	3ml 22G*1 1/4"
100064IM	3ml 23G*1 1/4"
100066IM	3ml 23G*1 1/2"
100068IM	3ml 25G*1 1/4"
100072IM	3ml 21G*1 1/2"
100074IM	3ml 20G*1"
100075IM	3ml 21G*1"
100076IM	3ml 22G*1 1/2"
100077IM	3ml 23G*1"
100078IM	3ml 25G*5/8"

REF	Size
100079IM	3ml 25G*1"
100084IM	3ml 20G*1 1/2"
100085IM	3ml 22G*1"
100059IM	3ml 18G*1 1/2"
100063SIM	3ml 23G*1" SIDE
100061SIM	3ml 21G*1 1/2" SIDE
100065SIM	3ml 23G*1 1/4" SIDE

SOL-MILLENNIUM[®]

REF	Size
140060IM	5ml 21G*1 1/4"
140062IM	5ml 22G*1 1/4"
140065IM	5ml 23G*1 1/4"
140072IM	5ml 21G*1 1/2"
140075IM	5ml 21G*1"
140076IM	5ml 22G*1 1/2"
140077IM	5ml 23G*1"
140084IM	5ml 20G*1 1/2"
140085IM	5ml 22G*1"
140061SIM	5ml 21G*1 1/2" SIDE
140076SIM	5ml 22G*1 1/2" SIDE


REF	Size
160062IM	10ml 22G*1 1/4"
160065IM	10ml 23G*1 1/4"
160072IM	10ml 21G*1 1/2"
160075IM	10ml 21G*1"
160076IM	10ml 22G*1 1/2"
160077IM	10ml 23G*1"
160084IM	10ml 20G*1 1/2"
160085IM	10ml 22G*1"
160061SIM	10ml 21G*1 1/2" SIDE

REF	Size
190061IM	20ml 21G*1 1/2" SIDE
190072IM	20ml 21G*1 1/2"
190075IM	20ml 21G*1" SIDE
190085IM	20ml 22G*1" SIDE

Technical information

	Component name	Material		
	LL Needle hub	PP: 5250T		
	O ring	Medical Silicone		
	Locating ring	HDPE: 2911		
	Gasket	TPE:181W-57		
	Barrel	PP: 5250T		
1. List of Materials	Locking ring	3ml/5ml/10ml PP: 5250T 20ml ABS: TR-558Al		
	Plunger	PP: 5250T		
	Needle cap	PP: RP344RK		
	Cannula	Stainless steel: SUS304		
	Needle hub	PP: H1500		
	Adhesive	Epoxy: YD-128		
	Needle Lubricant	Silicon oil: Xilikang 201-350cst		
	Barrel Lubricant	Silicon oil: DC 360 12500cst		
	2. Latex free	YES		
3. PHT / DEHP / PVC / BPA free	YES			
4. Shelf life	5 years			
5. Sterilization method	Sterilized using Ethylene Oxide			
6. Packaging specification	6.1 Sales unit	3ml\5ml\10ml 20ml	100 50	Units per box
		3ml 800		
		5ml\10ml 1200		
		5ml\10ml SIDE	800	Units per case
		20ml 600		
		20ml SIDE 600		

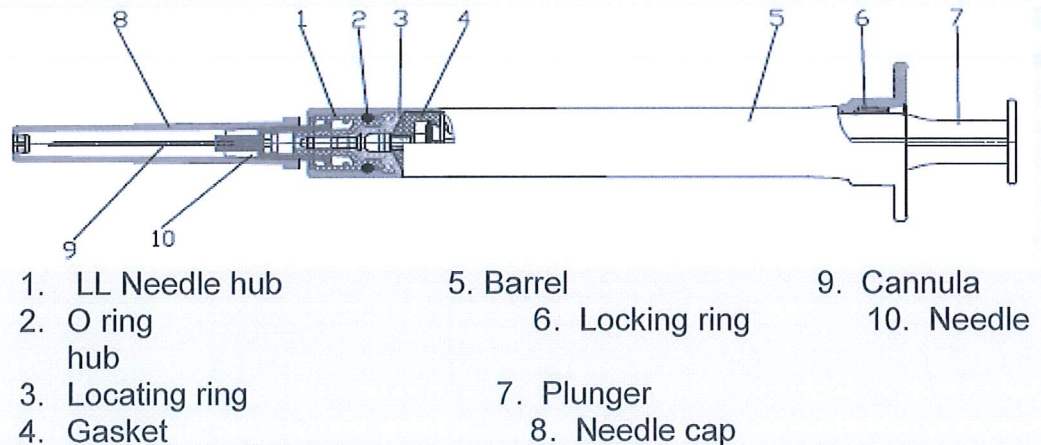7. Technical Drawing

Quality and Regulatory information

- | | |
|---------------------------|--|
| 1. Quality certificate | Quality Management System according ISO 13485 |
| 2. Product classification | Class IIa according to Annex IX of MDD 93/42/EEC |

The product is compliant with the following standards and regulations:

Document reference	Title
BS EN ISO 7886-1:1993	Sterile hypodermic syringes for single use-Part 1: Syringes for manual use
BS EN ISO 7864: 1993	Sterile hypodermic needles for single use
BS EN ISO 7886-4: 2006	Sterile hypodermic syringes for single use-Part 4: Syringes with re-use prevention feature
BS EN ISO 9626: 1991	Stainless steel needle tubing for the manufacture of medical devices
BS EN ISO 6009: 1992	Hypodermic needles for single use—colour coding for identification
ISO 594-1:1986	Conical fitting with 6% (Luer) taper for syringes, needles and certain other medical equipment part 1 general requirement
ISO 594-2: 1986	Conical fittings with a 6% (Luer) taper for syringes, needles and certain other medical equipment-part 2: Lock fittings
BS EN ISO 10993-1, 2006	Biological evaluation of medical devices
BS EN ISO10993-4:2002	Biological evaluation of medical devices—part 4

3. List of standards

SOL-MILLENNIUM[®]

	BS EN ISO10993-5: 2009	Biological evaluation of medical devices—part 5
	BS EN ISO10993-7:2008	Biological evaluation of medical devices—part 7
	BS EN ISO10993-10: 2010	Biological evaluation of medical devices—part 10
	BS EN ISO 15223-1:2008	Medical devices - Symbols to be used with medical device labels, labeling and information to be supplied
	BS EN ISO 11607-1: 2006	Packaging for terminally sterilized medical devices— Part 1
	BS EN ISO 11607-2: 2006	Packaging for terminally sterilized medical devices— Part 2
	BS EN ISO 11737-2: 1998	Sterilization of medical devices-Microbiological methods-Part 2: Tests of sterility performed in the validation of a sterilization process

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TO:	To whom it may concern
FROM:	Kate Liu Jing, Director, Quality Assurance
DATE:	

Sol-Millennium Medical Inc. is the legal manufacturer for all FARMATEXA® Sicurtouch branded products. We hereby declare, that they meet provisions of the Council Directive 93/42/EEC,

the products:

- are not made with natural rubber latex and any harmful substances;
- are latex, DEHP/PVC/BPA free
- are Phthalate free,
- are packed in a pyrogen-free, latex free (natural rubber-free) and recyclable packaging,
- shelf life is 5 years

the packaging (box, case) are:

- pyrogen free
- not made with natural rubber latex and any harmful substances;
- recyclable.

Kate Liu Jing,
Director, Quality Assurance
Sol-Millennium Medical Group

SOL-Millennium Medical Inc.
5415 Sugarloaf Parkway, Suite 2203 · Lawrenceville, Georgia 30043 USA
Office 404.973.2200 Fax 404.973.2212
www.sol-millennium.com

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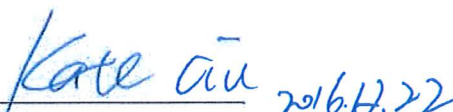
To whom it may concern

Date: 2016-12-22

Subject: Safety syringes FZ brand / Standard syringes FZ brand

We, Sol-Millennium Medical Inc. are the legal manufacturer for SOL-CARE™ and SOL-M™ branded products listed below:

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We hereby declare that our products are not made with natural rubber latex, DEHP/PVC free, the accelerated aging tests have been performed successfully for the above-mentioned products with Shelf life of 5 years.

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Omni Quality, s.a. Belgium
1000 Brussels, Belgium

Distributie in:
ARMACOM S.p.A.
MEDICOM S.p.A.
MEDICOM S.p.A.

XXXXHEVZ Made in China

EN Luer Lock Safety Syringe with Exchangeable Needle

Instructions for Use

1. Peel package as indicated to allow proper removal of SICURTUCH Luer Lock Safety Syringe with exchangeable needle and remove needle cap.
2. Prepare vial for medication draw up.
3. Penetrate center of vial with needle and draw up desired dose of medication. (See Figure 1).
4. Prepare injection site and change needle if desired.
5. Administer medication per facility protocol.
6. To engage one hand safety feature activation and deliver the full dose, push the plunger through the zero line. (See Figure 2)
7. Pull back on the plunger until the locking ring is engaged. Observe withdrawal of needle into the barrel. You can pull back and lock the plunger with one hand only. (See Figure 3)
8. Snap off plunger to permanently disable syringe, you can snap off the plunger with one hand only. (See Figure 4)

Single-handed safety mechanism activation with irreversible locking mechanism, confirmed with an audible click.

Caution:

- * This device is sterile unless the package has been damaged or opened.
 - * If unable to activate safety mechanism, discard syringe immediately into an approved sharps container.
 - * This device is single use only.
 - * Maximum needle length cannot exceed 1 1/2" or 38,1mm.
- Federal (USA) law restricts this device to sale by or on the order of a physician or other licensed practitioner.

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