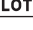
## Simboli / Symbole / Symboles / Symbolen / Symbols

	Fabbricante / Hersteller / Fabricant / Producent / Manufacturer
	Attenzione, leggere le istruzioni per l'uso / Achtung, Gebrauchsanweisung beachten / Attention, lire les documents avant emploi / Let op, lees de handleiding / Caution, read instructions for use
	Codice prodotto / Bestellnummer / Numéro de catalogue / Artikelnummer / Catalogue number
	Numero di lotto / Chargenbezeichnung / Numéro de lot / Lotnummer / Batch code
	Temperatura minima e massima di conservazione / Temperaturbegrenzungen / Limites de température / Temperatuurgrenzen / Temperature limitation
	Data di scadenza / Verwendbar bis / Date d'expiration / Houdbaar tot / Use by
	Questo prodotto è conforme alla direttiva 98/79/CE sui dispositivi medico-diagnostici <i>in vitro</i> / Dieses Produkt entspricht den Bestimmungen der Richtlinie 98/79/EG für <i>In-vitro</i> -Diagnostika / Ce produit est conforme à la Directive 98/79/CE sur les dispositifs médicaux de diagnostic <i>In-vitro</i> / Dit product voldoet aan de vereisten van de Richtlijn 98/79/EC voor medische apparaten voor <i>In-vitro</i> diagnostiek / This product fulfills the requirements of the Directive 98/79/EC on <i>In-vitro</i> Diagnostic Medical Devices
	Dispositivo medico-diagnostico <i>in vitro</i> / <i>In-vitro</i> -Diagnostikum / Dispositif médical de diagnostic <i>In-vitro</i> / Medisch apparaat voor <i>In-vitro</i> diagnostiek / <i>In-vitro</i> Diagnostic Medical Device
	Aggiunte o modifiche rilevanti rispetto alla precedente versione delle istruzioni / Wesentliche Ergänzungen oder Änderungen gegenüber der vorigen Version der Gebrauchsanweisung / Additions ou modifications significatives par rapport à la version précédente / Significante toevoegingen of wijzigingen t.o.v. de voorgaande revisie op de gebruiksinstructies / Significant additions or changes from previous instructions for use revision
	Contenuto sufficiente per <n> test / Ausreichend für / Suffisant pour / Voldoende voor / Sufficient for
	Indicato per l'autocontrollo / <i>In-vitro</i> -Diagnostikum für die Selbstkontrolle / Appareil <i>In-vitro</i> pour l'autocontrôle / <i>In-vitro</i> apparaat voor zelfcontrole / <i>In-vitro</i> device for self-testing
	Validità della soluzione di controllo dopo l'apertura del flacone in plastica: 90 giorni / Haltbarkeit der Kontrolllösung nach dem Öffnen des Plastikfläschchens: 90 Tage / Durée de conservation de la solution de contrôle après ouverture du flacon en plastique: 90 jours / Houdbaarheid controlevoelstof na openen van het plastic flesje: 90 dagen / Control solution lifetime after opening the plastic bottle: 90 days

## Disponibilità / Verfügbarkeit / Conditionnement / Beschikbare producten / Availability

	<b>46630</b> GLUCOFIX® TECH Control N: 1 flacone x 2,5 mL / 1 Fläschchen x 2,5 mL / 1 flacon de 2,5 mL / 1 flesje x 2,5 mL / 1 bottle x 2.5 mL
	<b>46631</b> GLUCOFIX® TECH Control H: 1 flacone x 2,5 mL / 1 Fläschchen x 2,5 mL / 1 flacon de 2,5 mL / 1 flesje x 2,5 mL / 1 bottle x 2.5 mL

 Data di pubblicazione: Gennaio 2019 / Ausgabedatum: Januar 2019 / Date de publication: Janvier 2019 / Uitgiftedatum: Januari 2019 / Date of issue: January 2019

 **A. MENARINI DIAGNOSTICS Srl**  
Via Sette Santi, 3  
50131 Firenze - Italy

Made in Taiwan

Per informazioni sulle soluzioni di controllo GLUCOFIX® TECH Control, contattare: / Wenn Sie Fragen zu der Kontrolllösung GLUCOFIX® TECH Control haben, wenden Sie sich bitte an: / Si vous avez des questions sur la solution de contrôle GLUCOFIX® TECH Control, veuillez contacter: / Neem voor eventuele vragen over GLUCOFIX® TECH Control a.u.b. contact op met: / If you have any question about GLUCOFIX® TECH Control, please contact:

 **IT**  
Distribuito da: / Vertrieb durch: / Distribué par:  
**A. MENARINI Diagnostics S.r.l.**  
Via Lungo L'Enza, 7  
50012 Bagno a Ripoli  
Firenze (Italia)  
Servizio Clienti 800 869110  
www.menarindiagnosics.it

 **NL**  
Gedistribueerd door:  
**MENARINI Benelux NV/SA**  
**Diagnostics Division**  
De Haak 8  
NL-5555 XK Valkenswaard  
Tel. +31-(0)40 2082000  
Gratis nummer: 0800 0225422

 **FR**  
Distribué par:  
**A. MENARINI Diagnostics France S.A.S.U.**  
3-5, rue du Jura - BP 70531  
F-94633 Rungis cedex  
Tel. +33 (0)1 56 34 69 10

 **UK**  
Distributed by:  
**A. MENARINI DIAGNOSTICS LTD**  
405 Wharfedale Road  
Wokingham - Berkshire RG41 5RA  
Tel. +44-1189-444100  
GlucoMen® Careline:  
(UK) 0800 243667  
(Republic of Ireland) 1800 709903  
(Northern Ireland) 0800 7837286  
Email: myglucomen@menarindiag.co.uk  
www.glucomen.co.uk

# GLUCOFIX® TECH Control

46630 - 02/20

- Soluzioni di controllo glucosio  
*Istruzioni per l'uso*
- Glukosekontrollösungen  
*Gebrauchsanweisung*
- Solutions de contrôle de glucose  
*Instructions for use*
- Glucose-controlevoelstoffen  
*Instructions for use*
- Glucose Control Solutions  
*Instructions for use*

 **0123** 

 **A. MENARINI**  
diagnostics

## ITALIANO

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## ENGLISH

Before use, carefully read these instructions, the User Manual of your meter and the GLUCOFIX® TECH Sensor Instructions for Use. If you have any questions, please contact the A. Menarini Diagnostics Customer Service.

### Intended use

The GLUCOFIX® TECH Control solution is only for use with the GLUCOFIX® TECH blood glucose meter or the GLUCOFIX® TECH 2K blood glucose and  $\beta$ -ketone meter. GLUCOFIX® TECH Control is designed to check that the meter and the GLUCOFIX® TECH Sensor test strips work properly. GLUCOFIX® TECH Control is an *in vitro* diagnostic medical device intended for self-testing (only for use outside the body).

### When to perform a control test

Carry out a control test if:

- You suspect the meter or test strips are not working properly;
- The meter has been dropped;
- The meter is damaged;
- Your test results do not match how you feel;
- You want to check the performance of the meter and test strips when you first get them and any time you want to check their performance before a blood glucose test.

### Contents

The GLUCOFIX® TECH Control sales package contains a plastic (low density polyethylene) dropper bottle of control solution with dropper tip, coloured screw cap (green for Level N and blue for Level H) and instructions for use. The dropper bottle contains either 2.5 mL of Level N control solution or 2.5 mL of Level H control solution. Both control solutions consist of D-glucose and other non-reactive substances (preservatives, colourants, viscosity regulating agents). The glucose concentration was assessed by means of hexokinase G-6-PDH method (secondary reference measurement procedure). The analyzer used to run the procedure was calibrated using a series of glucose standards (primary calibrators) that is traceable to NIST SRM917C standard.

### Storage and usage conditions

- Store the control solution at a temperature of 4-30 °C (39.2-86 °F). Do not freeze or refrigerate; avoid exposure to heat and direct sunlight.
- Colourants in the control solution may stain clothing or skin.
- **Do not use** the control solution after its expiry date.
- **Do not use** the control solution for more than 3 months after first opening the bottle. We recommend to write the discard date (opening date + 3 months) on the label.

### Warning and safety information

- Keep the product out of the reach and sight of children. Small items may represent choking hazards.
- The used control solution bottle must be disposed of according to local regulations.
- The product is not intended for human consumption, injection or skin/body application.

### Testing Procedure

#### Materials Provided.

GLUCOFIX® TECH Control (level N or H).

#### Materials required but not provided.

Meter, GLUCOFIX® TECH Sensor test strips.

For more details see the meter's user manual.

For accurate test results, allow the control solution, meter and test strips to adjust to the surroundings for at least 30 minutes before performing a control solution test. Operating conditions are: temperature 5-45 °C (41-113 °F) and relative humidity 20-90%.

1. Insert a new GLUCOFIX® TECH Sensor test strip into the test strip port. The drop icon will start blinking on the screen of the meter.

2. To enable the CTL mode, please refer to the respective paragraph in the User Manual of your meter.

- If that mode has not been enabled before performing the control solution test, the result will be stored in the meter as a blood test and used for calculating the averages.

3. Gently shake the control solution bottle before testing. Discard a drop before use. Squeeze a drop of control solution onto a clean, hard, dry surface.

4. **Touch the drop of control solution with the tip of the test strip** until the check window of the test strip is full.

- The test will not start if you apply the control solution directly to the check window. The test starts when the check window is full.
- Tightly close the control solution bottle immediately after use.

5. **The test result will be displayed on the meter screen.**

6. **Press the release button to remove the test strip.** The meter will switch off.

### Expected Values

Check that your test result is within the acceptable range shown on the vial label. If within this range, the meter and the test strips are working properly. If out of range, repeat the control solution test again.

If you still have test results that fall outside the acceptable range, stop using the system, and contact the A. Menarini Diagnostics customer service.

A. MENARINI DIAGNOSTICS S.r.l.  
Via Sette Santi, 3  
50131 Firenze - Italy

<b>Description</b>	GLUCOFIX® TECH Control Insert - IT/FR/NL/UK		<b>Colours Used</b>
<b>Code</b>	46658		I - BLACK C
<b>Rev</b>	02/20		
<b>Size (mm)</b>	w 200 x h 280		

<b>Edition</b>	1	2	3	4	5	6	7	8
<b>Date preparation</b>								

**PLEASE READ THIS IMPORTANT INFORMATION:** Please ensure this proof matches your Artwork requirements. Please check ALL aspects of the proof i.e. text, font, spelling, colours, size, construction, copy position, barcode, pharma codes, orientation of graphics etc.

**PLEASE REFER TO AGREED COLOUR STANDARDS / PANTONE REFERENCE FOR COLOUR MATCH**

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Document Number	Version	Status	Effective Date*
PAL-000034250	3.0, CURRENT	Effective	03-Aug-2020
Previous Doc Number		Author Name	
		Giorgetti Nicola	

User	Date	Justification
Tugnoli Monica	03-Aug-2020 08:47:40 (GMT)	QA Approval
Sicurani Marco Antonio	03-Aug-2020 08:51:44 (GMT)	Manager Approval

\* based on server time zone